

# The Partnership of Connectivity, Traceability and Productivity

## DXT® Data Management System for CompoMat® G5 Plus

**Secured two-way interface** between device and customers' computer system

---

Potential to **improve productivity** with simplified workflow, error reduction and reduction of waste

---

Data may be used to aid in **product suitability** and **quality investigations**

---

Information to support **customized training curriculums**

---



# DXT Data Management System

Connecting DXT to your CompoMat G5 *Plus* and CompoMaster Net G5 Plus system provides automated documentation to meet your center guidelines for workflow traceability and drive productivity within your operations.


PROCEDURE REPORT			
Donation ID	A99999999999	Instrument Type	CompoMat G5
Program Number	3	Preparation Mode	No
Program Name	HRD QUAD BF	Instrument Serial Number	8CPT9999
Operator ID 1	004	Region	CompoMat Sites
Operator ID 2	-	Site	Green Valley
Process Date	1/6/2021	Pause Time	00:00:00
Process Start Time	2:48:52 PM	Number of Pauses	0
Process End Time	2:50:25 PM	Result	-
Process Total Time	00:01:33		
Counter Start Time	-		
Counter End Time	-		
Counter Difference Time	-		
SCALE WEIGHTS			
Plasma Scale (g)	Press Scale (g)	RCC Scale (g)	
330	-	-	
-	-	-	
-	-	-	
-	-	-	
-	-	-	
BARCODE DATA			
Reference Number	-	Additional Barcode	-
Batch Number	-	Dummy Barcode	-
Centrifuge	-	Batch 1	-
		Batch 2	-
INCIDENCE DATA			
No incidence data exists for this procedure			

## Automated Procedure Record provides support to reduce waste

- DXT stores an electronic copy of the procedure results, providing complete and accurate documentation, including scanned values.
- With automated documentation, DXT offers the ability to reduce product discards while eliminating procedure record rework due to manual documentation and data entry errors.

## Procedure Listing provides insights to identify areas to increase overall product yields

- Real time access to procedure listings is provided with a historical view of procedures from all devices.
- Filters allow status visibility and enable improvements on key performance indicators. Filter by program, operator, device and more.

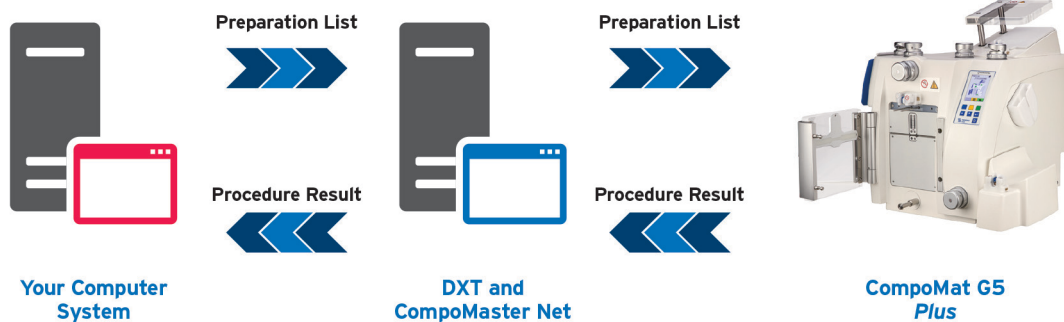

Software application designed to receive, store and transmit procedure information

Secured two-way interface between device and customers' computer system

Supports connectivity with Amicus Separator, AmiCORE Apheresis System, Alyx Component Collection System, CompoGuard, Aurora Plasmapheresis System and CompoMat G5 *Plus* devices

Contact your Fresenius Kabi representative for more information regarding DXT and the potential to interface with your center's blood bank system.

Certain devices, software versions and protocols may not be available depending on approval or clearance by the appropriate regulatory agencies in each country. Contact a Fresenius Kabi representative for questions on the regulatory status in your country.

Refer to the DXT Operator's Manuals and the CompoMat G5 *Plus* Operator's Manual for a complete list of warnings and precautions associated with the use of these products.

This marking reflects compliance with the applicable CE Marking requirements for the CompoMat G5 *Plus* medical device.

